



# Test instructions for **BFP**<sup>™</sup> Pregnancy Test Strips

## Instructions

1. The BFP Pregnancy Test Strip can be performed at any time of the day; however, testing first morning urine is preferable.
2. Urinate into a clean, dry cup or container.
3. Remove the test strip from the pouch.
4. Hold the test strip in a vertical position with the arrows pointing downward.

Dip the test into the urine approximately 1/4 inch and hold it there for at least 5 seconds. Do not dip past the stop line.

5. Remove the test strip and lay it down flat. Wait 10 minutes.

## Reading Your Results



While you are waiting for your result, you may notice a colored solution moving across the test strip. This is normal. You need to wait 10 minutes to confirm the result. The test should not be evaluated after 30 minutes. The exact color of the line is not important.

**1 line = Not Pregnant**

**2 lines = Pregnant**

If only 1 colored line appears as the control line, the test is negative and you can assume you are not pregnant.

If 2 colored lines appear, 1 as the testing line and 1 as the control line, the test is positive and you can assume you're pregnant. Even if the lines are very light in color, you can still assume you are pregnant.

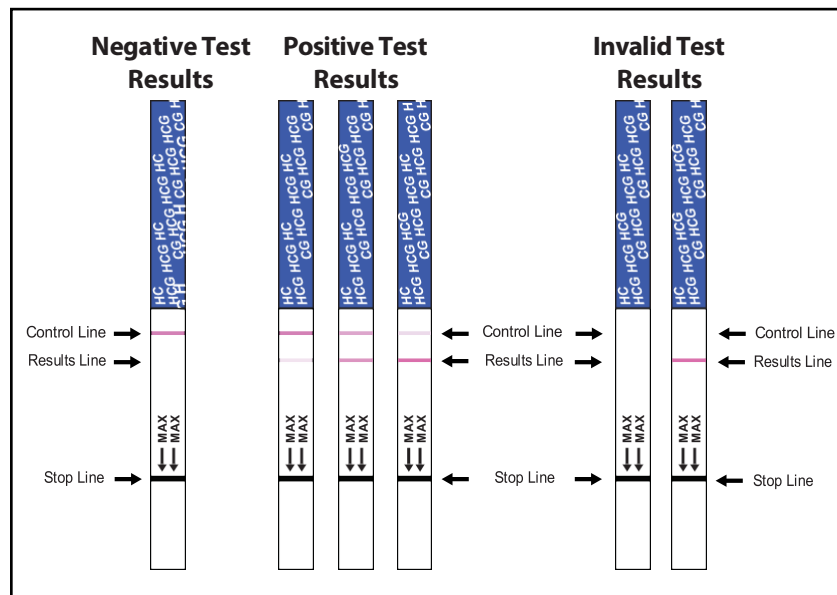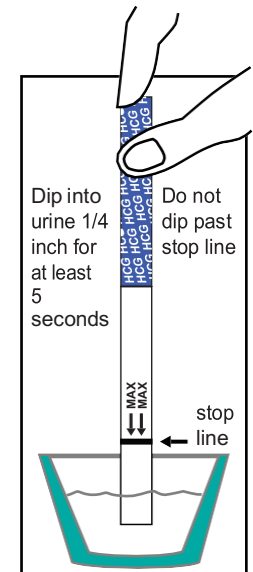

**Invalid Results:** If the control line does not appear, the test is invalid. Invalid results may be an indication that the test strip was not thoroughly saturated in urine. In this case, re-dip the test for at least 5 seconds and wait 10 minutes to read results. If there is still no control line, the test is invalid and you should retest the following day. Each test is designed for a single use and should not be reused.



**FAIRHAVEN HEALTH**

- Ovulation Prediction Tools
- Fertility Supplements
- Pregnancy & Ovulation Tests
- And Much More!

[www.fairhavenhealth.com](http://www.fairhavenhealth.com)

[www.ovagraph.com](http://www.ovagraph.com)